

Take one — they're free

If you use a mask with elastic bands around your ears, these clips can help release pressure on your ears.

Cheetah™ 1.0 Flexible Mask Clip Instructions

Design by Roy Leban


Before first use, remove masking if present
Clean with soap/disinfectant as necessary.



Mask straps can go around or under ears, whichever is more comfortable. You will have to experiment to find the best position for you from the six possibilities. The clip should hold the mask snugly but not dig into your neck.

1 The tabs are intended for permanent attachment to your mask. Be careful to avoid breaking any tabs while determining the best adjustment, then leave clip attached to mask. If you are cycling through multiple masks, use a different clip for each one.

2 To put mask on, hold clip in your non-dominant hand and opposite mask strap in your dominant hand. Put on mask, then attach strap to chosen slots. To remove mask, release strap from the slots only.

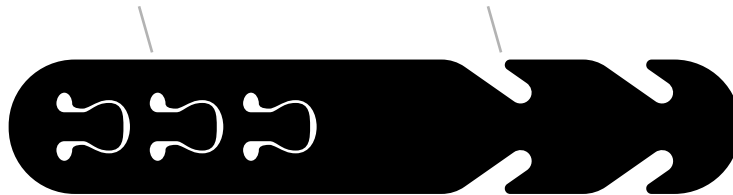

Note: On both sides, the part of the strap secured by the tab or between the sides of the slots should be underneath the clip (closer to your neck).

© Copyright 2020 by Roy Leban